

Earlier and Accurate


Bio-Med Midstream Pregnancy Test

- More than 99% Accurate in Laboratory Studies and Practical Usage
- Asses as Early as 4 Days Before Missed Period
- Easy-to-Read Results in 3 Minutes
- Simple one step test with urine

Manufactured by:
Bio-Med USA, Inc.
P.O. Box 6173
Paterson, NJ 07509
Made in USA

BEFORE YOU BEGIN


- Read the instructions carefully.
- Do not open the pouch until you are ready to start.
- Store the test stick at room or refrigerated temperature (2°-30°C, 36°-86°F) while not in use.
- For *in vitro* diagnostic use only.
- Do not use this test after the expiration date printed on the pouch.
- Do not reuse this test unit.

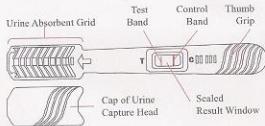


INSTRUCTIONS

- This test stick should be brought to room temperature prior to assay if it was stored at refrigerated condition.
- Open the foil pouch by tearing along the split. Remove the test stick from the pouch. As pictured, hold the round end of

the stick. Use another hand to remove the head cap to expose the urine absorbent grid.

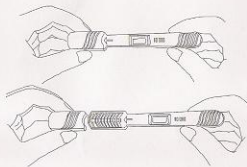


- Hold the test by the thumb grip with the exposed Absorbent Grid pointed downward. Urinate only on the Grid for 5 to 10 seconds until grid area gets wet thoroughly. Do not urinate on the window. Keep grid at pointing down position for

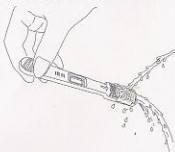

several seconds to get rid of the extra urine naturally. Do not shake away the urine trapped on the grid. You may collect the urine in a clean, dry

- container and dip only the Grid (the area below the maximum pointed by the arrow) in the urine for 5 seconds.
- Replace the cap. Place the test on a clean, level surface with the window facing upward while you are waiting for the test result. If you wish, you may leave the Grid exposed. As the test begins to work, you will see a light pink color stream moving across the window.

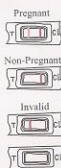

READING THE RESULTS

- Wait 3 minutes to read the result. After 3 minutes, a purple band will appear in the control area to show you the test is completed.
- If there is any purple band in the test area, you are pregnant. If there is no band in test area, you are not pregnant. As long as you find a band in the control area, the reading is valid. It does not matter if one band is fainter than the other.
- Do not read the result after 5 minutes.

QUESTIONS AND ANSWERS

- Q. How does the test work?
- A. When a woman gets pregnant, her body produces a pregnancy hormone called hCG (human Chorionic Gonadotropin) in her urine. Certain Midstream Pregnancy Test detects the presence of the hormone from first day of missed period.
- Q. What do I do if the test is positive?
- A. You should see your doctor who will discuss your pregnancy with you and explain what steps you should take next. It is recommended that you retest three days later by using another Certain Midstream Pregnancy Test. The earlier you see your doctor, the better it is for your health and the health of your baby.
- Q. What do I do if the test is negative?
- A. A negative result means that no pregnancy hormone has been detected and you are probably not pregnant. If your period does not start within a week, you may have miscalculated the day your period was due. Retest by using another Certain Midstream Pregnancy Test. If the test still gives you a negative result and you still have not menstruated, you should see your doctor.
- Q. Do I have to test in the morning?
- A. No, Certain Midstream Pregnancy Test can be used at any time of the day from the day your period is due.
- Q. What if the test is allowed to run for more than 5 minutes?
- A. A positive (Pregnant) test result will not change after the test is finished. A negative (Not Pregnant) test result should be read as soon as possible at 3 minutes. Over time, some negative results might turn to a weak positive result because the test chemicals keep on reacting.
- Q. How accurate is the Certain Midstream Pregnancy Test?
- A. In laboratory studies, this test was shown to be over 99% accurate.
- Q. Can any medicines affect the result?
- A. Only medicines which contain hCG can give a false reading. HCG-containing drugs include: A.P.L., Pregnyl, Follutein and Profasi. Hormone therapies (such as Clomid, Serophene, or Pergonal), painkillers, or the contraceptive pill should not affect the test result.
- Q. Can I get a false or misleading result?
- A. This should not happen if you follow the instructions correctly. However, if you get the capture head wet before using the test, if the test is out of the foil pouch for more than 24 hours before you conduct the test, if you don't get enough urine on the capture head, or if the test is not kept with the capture head pointing down when immersed in urine, then you may not get a reliable result. Certain rare medical conditions can also cause a false result. If you repeatedly get unexpected results, you should check with your doctor.
- Q. How soon can I use Certain Midstream Pregnancy Test?